

Masunaga model – Tango/Odette/Audrey

Dear Optical Dispenser,

As you may be aware, the Acetate rims of this type of frame can shrink and become loose within the metal rims. (Acetate material can shrink over time).

In order to achieve the best dispensing result, we recommend to cut the lenses to the size of the provided lens pattern (and not traced to the acetate rim) then to carefully push the lens into the acetate rims which will stretch to accommodate the lens snugly.

Example for Tango:

The Tango Lens pattern is at indicated size 47 + 1mm. This means 48mm including V cut facet.



Please do not hesitate in contacting us should you require further information.

Best regards

Noo Eyewear